

luxiva®

Luxiva Skin Mesotherapy Solution Injection Protocol



☎ 0942 600 1516

✉ info@atrastaray.com

🌐 atrastaray.com

🌐 luxiva.ir

Scan to Direct Call



Objective

To rejuvenate the skin, improve elasticity, and reduce the appearance of fine lines and wrinkles using Luxiva for facial mesotherapy.

Required Materials

-  Luxiva Skin Mesotherapy solution
-  Mesotherapy injection device (e.g., mesogun, microneedling pen, or syringe with a fine needle)
-  Antiseptic solution (e.g., chlorhexidine or alcohol swabs)
-  Sterile gloves
-  Cotton pads or gauze
-  Topical anesthetic cream (optional)

Patient Preparation

Consultation

Conduct a thorough consultation to assess the patient's skin type, medical history, and suitability for mesotherapy.

Informed Consent

Explain the procedure, potential risks, and benefits. Obtain written informed consent.

Skin Preparation

Cleanse the patient's face thoroughly with a gentle cleanser.

Anesthesia (Optional)

Apply a topical anesthetic cream to the treatment area. Leave it on for 20–30 minutes, then remove and cleanse the area again.

Procedure

- 1 Positioning:** Ensure the patient is comfortably lying down or seated in a position that allows easy access to the treatment area.
- 2 Antisepsis:** Disinfect the treatment area with an antiseptic solution.
- 3 Loading the Device:** Load the Luxiva solution into the mesotherapy injection device or draw it into the syringe.
- 4 Injection Technique:**
 - **Manual Injection:**
 - Use a fine needle (26–32G).
 - Inject small amounts of the solution intradermally at intervals of 1–2 cm across the entire face.
 - **Mesogun or Microneedling Device:**
 - Set up the device according to the manufacturer's instructions.
 - Use the device to administer the solution uniformly across the face, adjusting the needle depth as needed (usually between 0.5–1.5 mm).

- 5 Massage:** After injection, gently massage the treated areas to ensure even distribution of the product.

6 Post-Treatment Care:

- Apply a soothing cream or gel to reduce redness and inflammation.
- Advise the patient to avoid makeup, strenuous exercise, and direct sun exposure for at least 24 hours.
- Recommend the use of a broad-spectrum sunscreen daily. After injection, gently massage the treated areas to ensure even distribution of the product.

Post-Treatment Instructions



Immediate Aftercare: Instruct the patient to keep the treated area clean and to avoid touching it excessively.



Follow-Up: Schedule follow-up appointments to monitor progress and to plan additional sessions if needed. Typically, a course of 3–6 sessions spaced 2–4 weeks apart is recommended for optimal results.

Possible Side Effects

- Redness and swelling at the injection site
- Bruising
- Mild pain or discomfort
- Temporary bumps or lumps under the skin

Emergency Protocol

In case of severe allergic reaction or anaphylaxis, administer appropriate emergency treatment (e.g., epinephrine) and seek immediate medical assistance.

Notes

- Always use sterile, single-use needles and devices to prevent infection.
- Adjust the treatment protocol based on the patient's specific skin needs and response to previous sessions.

luxiva®

